



PRE-CLEANING FOLLOWING THE ENDOSCOPIC PROCEDURE

The ENDOSS Pre-Clean Kit enables immediate pre-cleaning of flexible endoscopes and other medical instruments. Bedside cleaning prevents organic matter from drying and supports effective reprocessing.

The kit contains a pre-dosed cleaning sponge with **Dr. Weigert MediClean Advanced** concentrate and a resealable pouch with a 500 ml fill indicator. Adding water instantly creates the correct cleaning solution for an efficient and reproducible process.

Why choose the Pre-Clean Kit?


- Hygienic
- Reproducible
- Ready for immediate use after endoscopy
- Enhances staff safety
- Compact and easy to use
- Suitable for various flexible endoscopes and other instruments
- Optimal first step in pre-cleaning
- Compatible with **Dr. Weigert MediClean Advanced** detergent

Easy - Efficient - Effective

Easy as 1-2-3

- 1 **Fill** - Add water up to the 500 ml fill line.
- 2 **Activate** - Squeeze the pre-dosed cleaning sponge several times to release the enzymatic cleaning concentrate.
- 3 **Clean** - Wipe the exterior of the endoscope and suction the solution through the channels according to the manufacturer's instructions.

INSTRUCTIONS FOR FLEXIBLE ENDOSCOPES

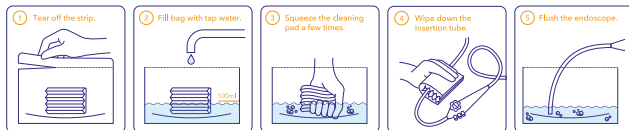

Application

Suitable for immediate pre-cleaning after procedures involving, among others:

- Gastrointestinal endoscopes
- Bronchoscopes
- ENT endoscopes
- Urological endoscopes
- Gynecological endoscopes
- Other compatible reusable medical instruments

Applicable within endoscopy departments, OR, outpatient clinics, wards, and other health-care environments where point-of-use cleaning is required.

Why enzymatic pre-cleaning?

During an endoscopic procedure, blood, mucus, proteins, fats, carbohydrates, and micro-organisms remain in and on the endoscope. Water dilutes this contamination but does not actively break it down. Therefore, international guidelines such as **ESGE**, **ESGENA** and **SGNA** recommend starting bedside pre-cleaning immediately after the procedure. This reduces the microbial load, prevents organic matter from drying, and contributes to effective reprocessing.



The power of enzymatic cleaning with Dr. Weigert

The Pre-Clean Kit contains a pre-dosed enzymatic cleaner that helps break down biological contamination before it can dry.

- Proteases help break down proteins.
- Lipases help loosen fats.
- Amylases help break down carbohydrate contamination.
- Helps keep contamination moist during transport.
- Optimal transition to pre-cleaning and automated cleaning & disinfection.

Every minute counts

Organic matter starts to dry immediately after the procedure. The **Pre-Clean Kit** activates the cleaning process immediately at the patient's bedside. Thanks to the pre-dosed enzymatic cleaner, proteins, fats, and other biological contaminants are loosened before they can adhere to the channels and surfaces of the endoscope. This supports consistent, reproducible, and guideline-compliant point-of-use cleaning.

Pre-Clean Kit

Art. No.: 80014

Packaging: • 1 pre-dosed cleaning sponge with MediClean Advanced Concentrate
• 1 resealable pouch with 500 ml fill indicator

Order unit: box of 50 pieces

Endoss B.V.

Ambachtsweg 4
4854 MK Bavel
The Netherlands

+31 - (0)76 - 53 09 268
info@endoss.nl
www.endoss.nl



How to order

Order via order@endoss.nl
Free delivery within the Netherlands.